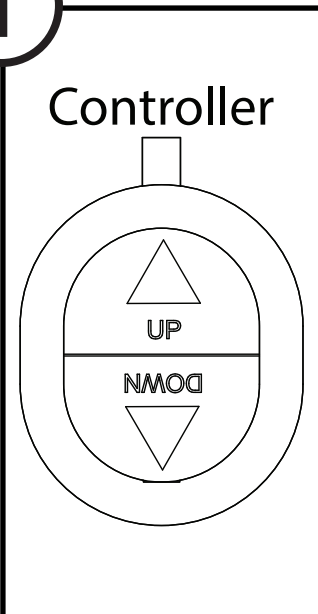
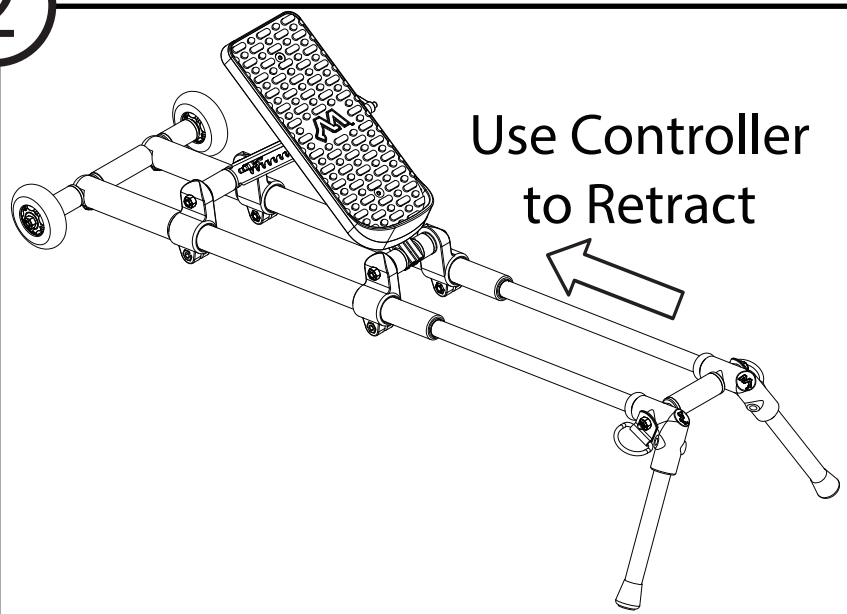
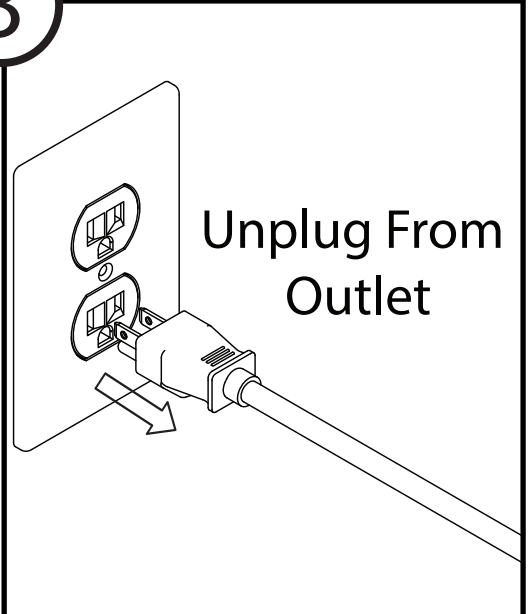
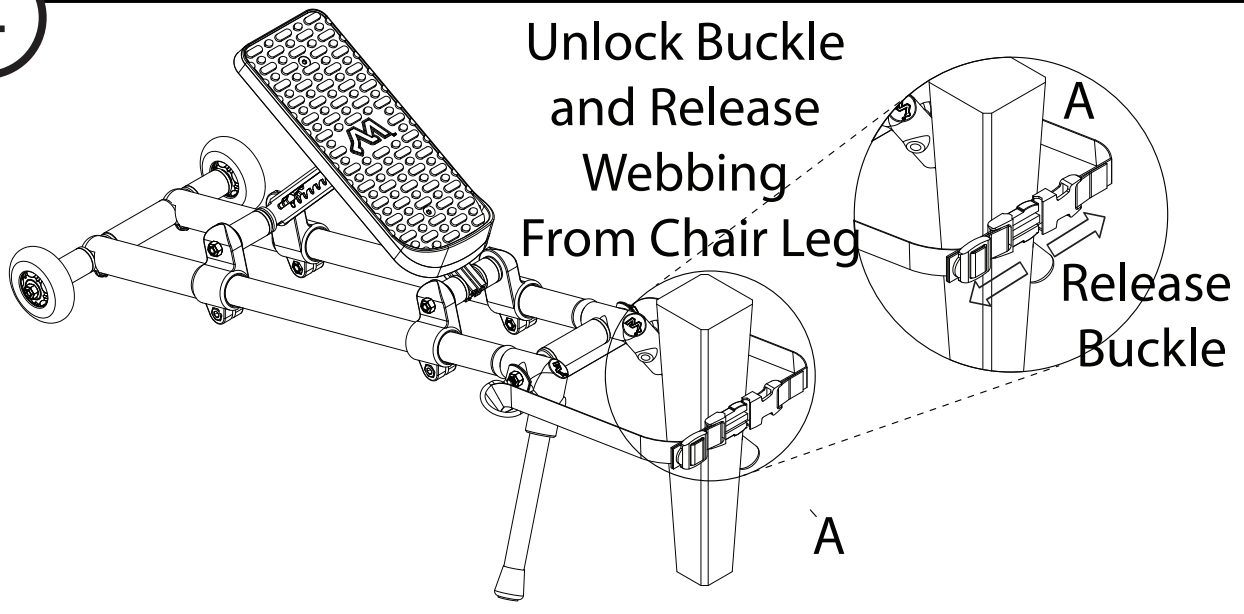


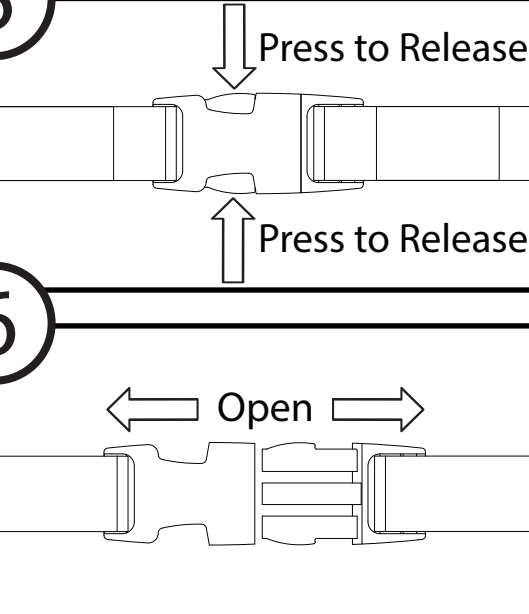
# Better Knee™ Return Instructions

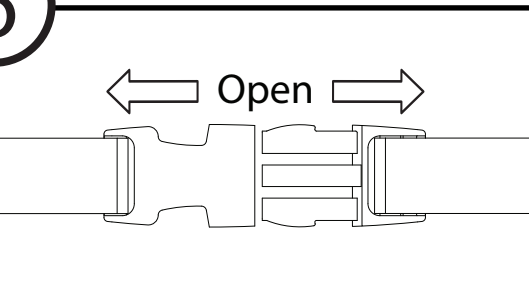
- 

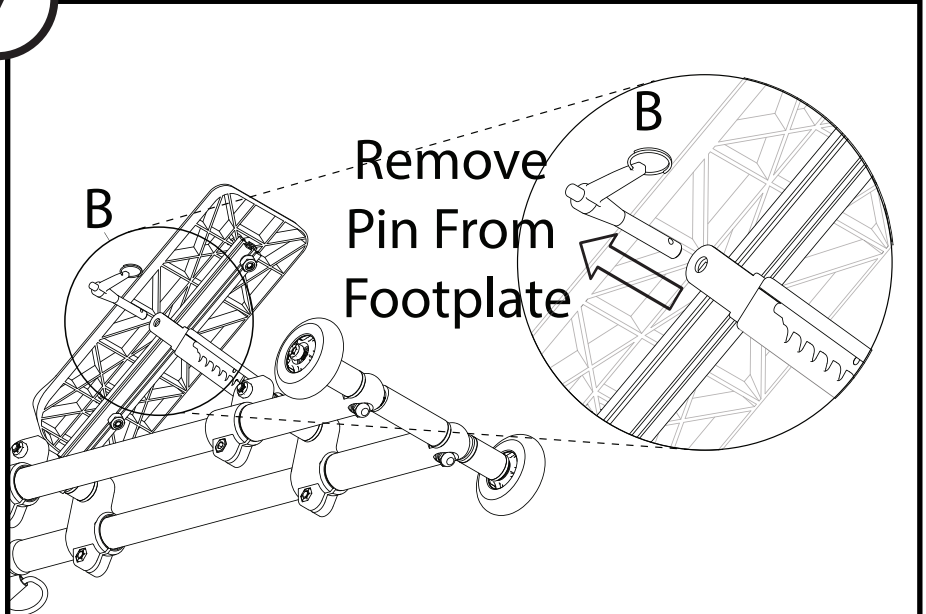
1 Controller
- 

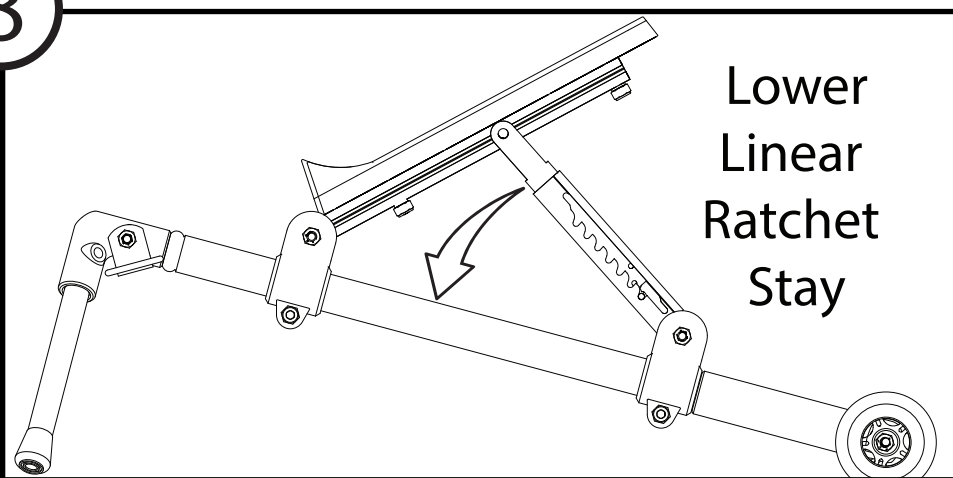
2 Use Controller to Retract
- 

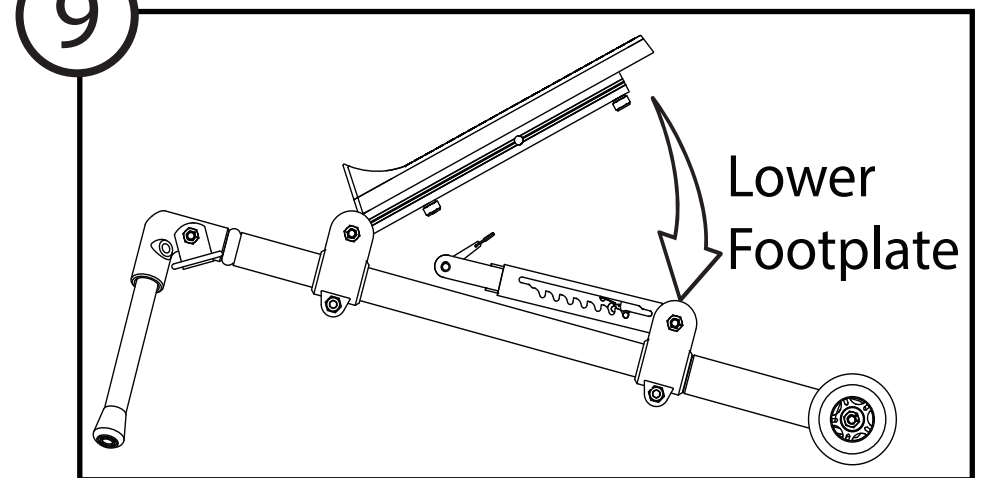
3 Unplug From Outlet
- 

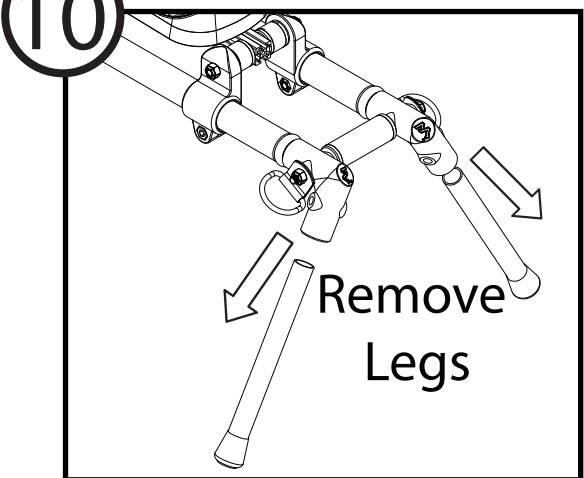
4 Unlock Buckle and Release Webbing From Chair Leg
- 

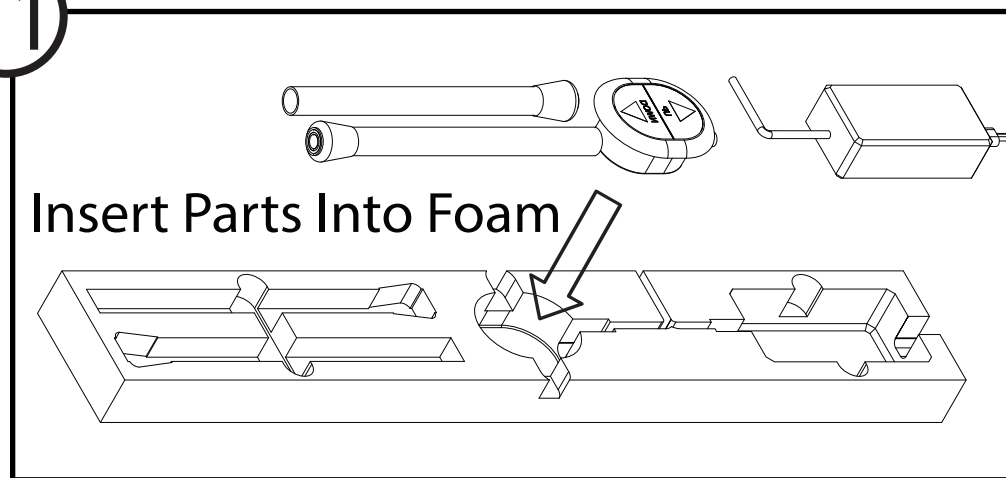
5 Press to Release
- 

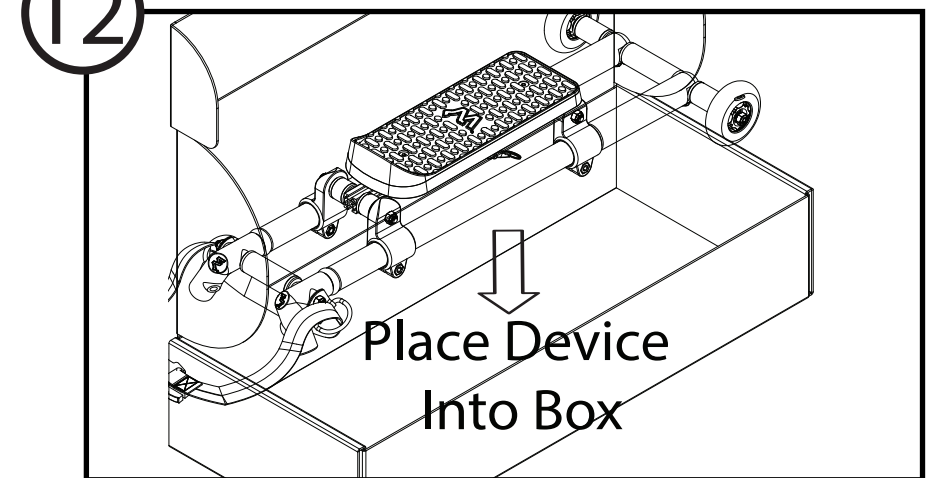
6 Open
- 

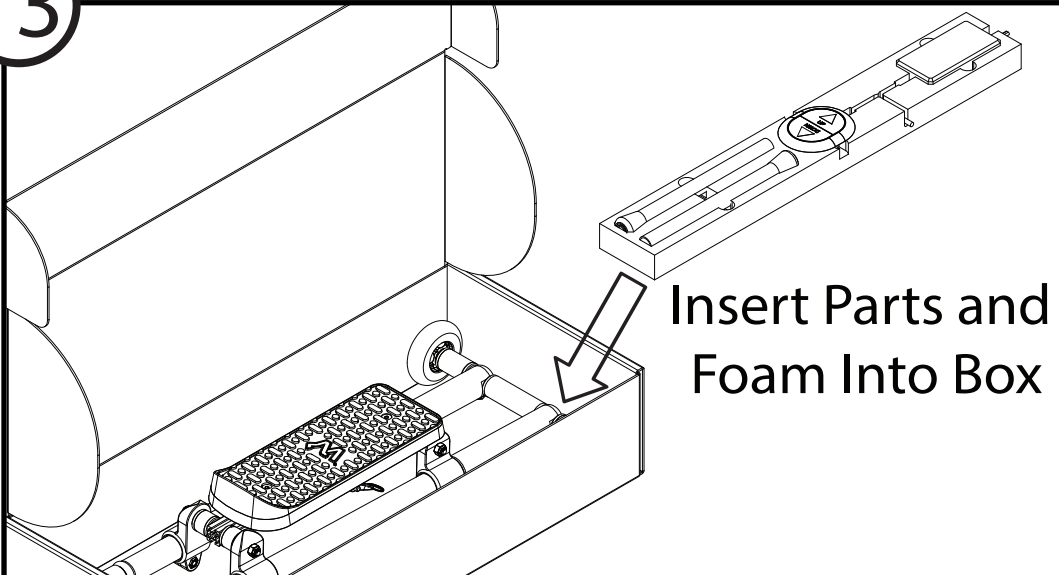
7 Remove Pin From Footplate
- 

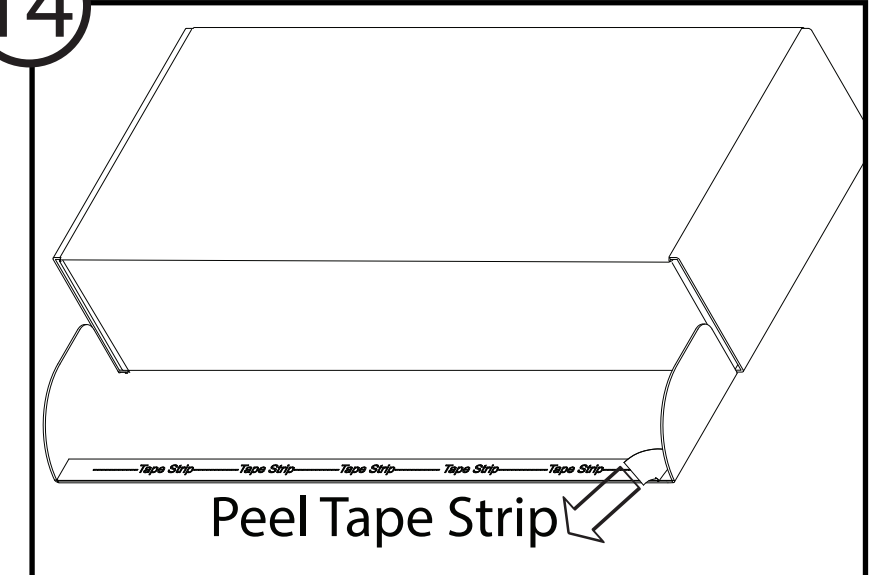
8 Lower Linear Ratchet Stay
- 

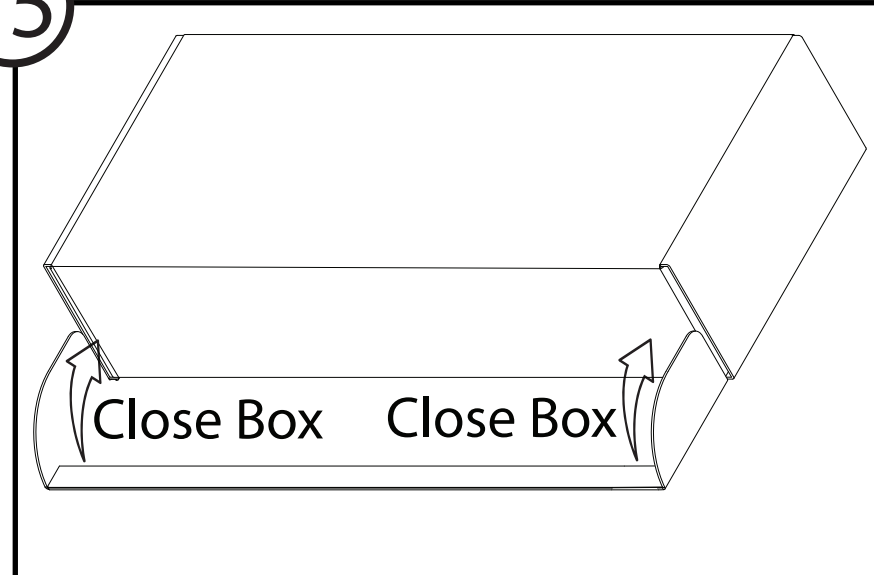
9 Lower Footplate
- 


10 Remove Legs
- 


11 Insert Parts Into Foam
- 


12 Place Device Into Box
- 

13 Insert Parts and Foam Into Box
- 

14 Peel Tape Strip
- 

15 Close Box
- 

16 Peel Off Top Layer of Return Label to Reveal Return Shipping Label
- 

17 **The UPS Store™**   
Drop off at the nearest UPS Store.