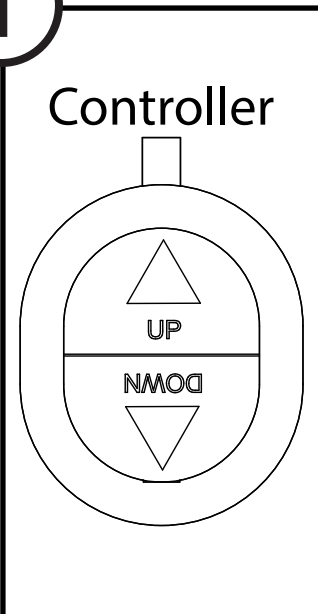
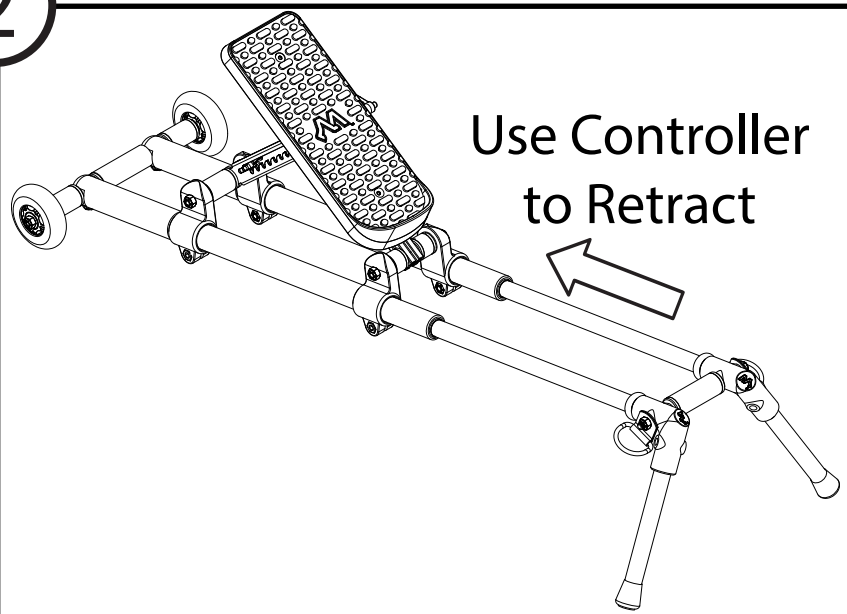
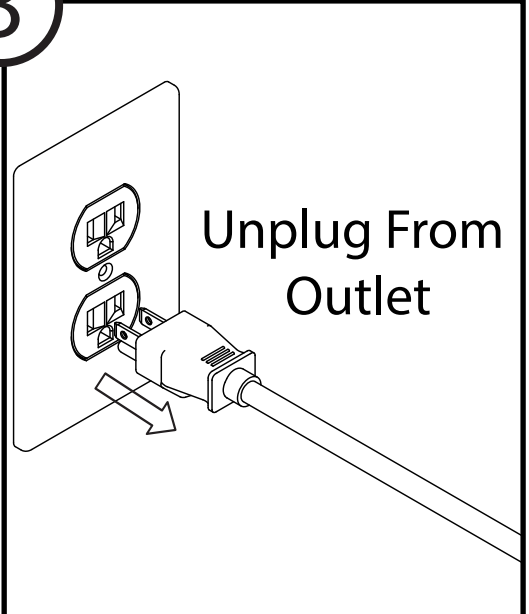
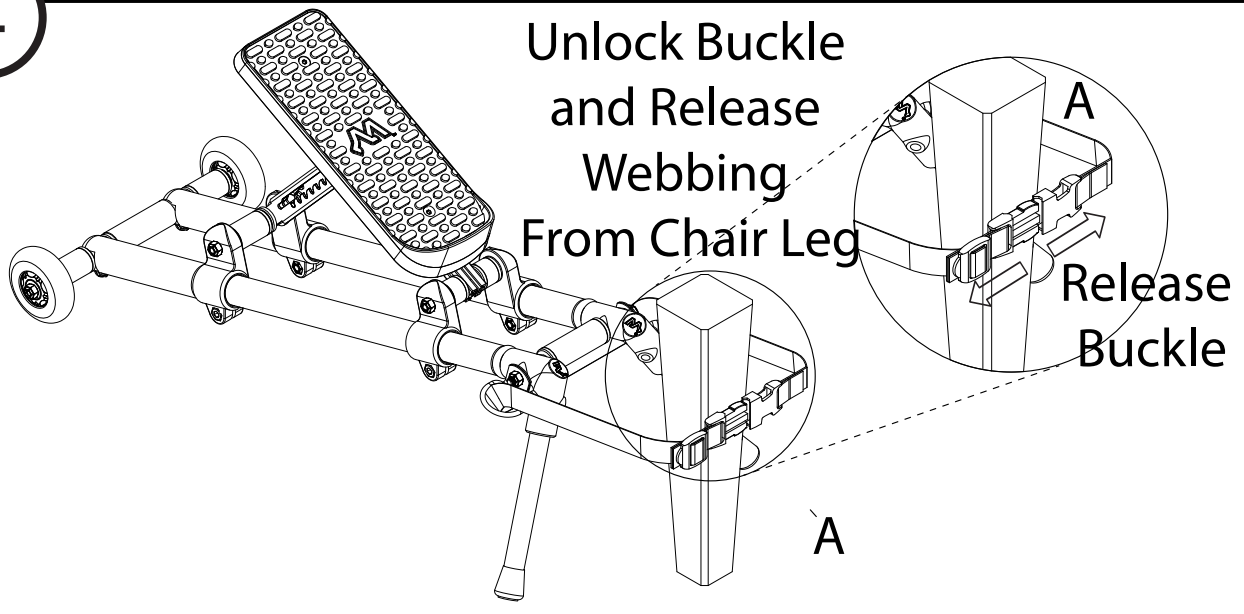


# Better Knee™ (Flexion) Return Instructions

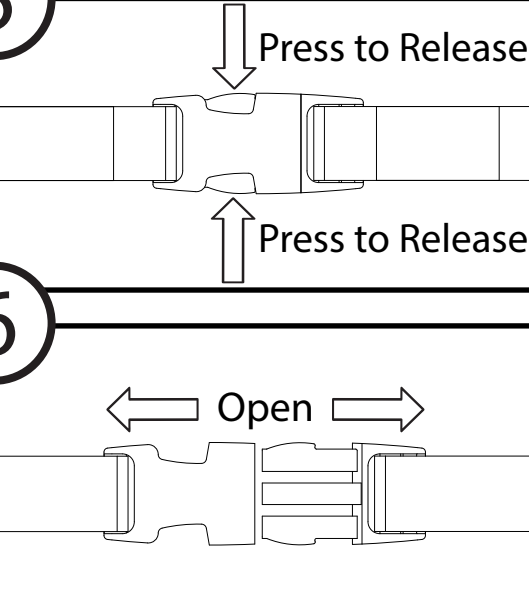
- 

Controller
- 

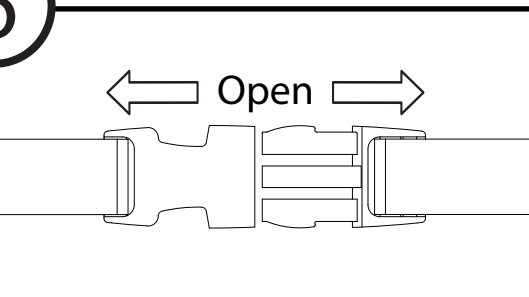
Use Controller to Retract
- 

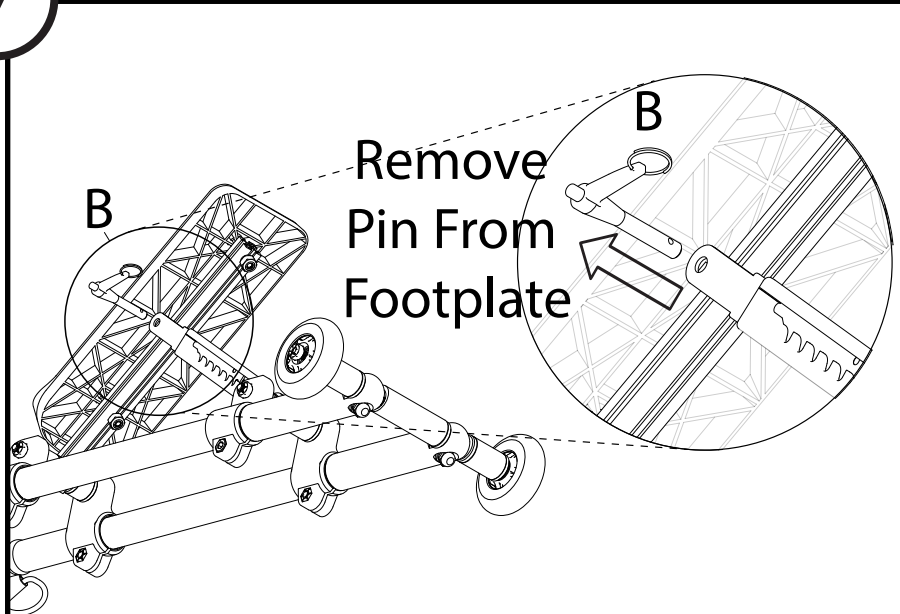
Unplug From Outlet
- 

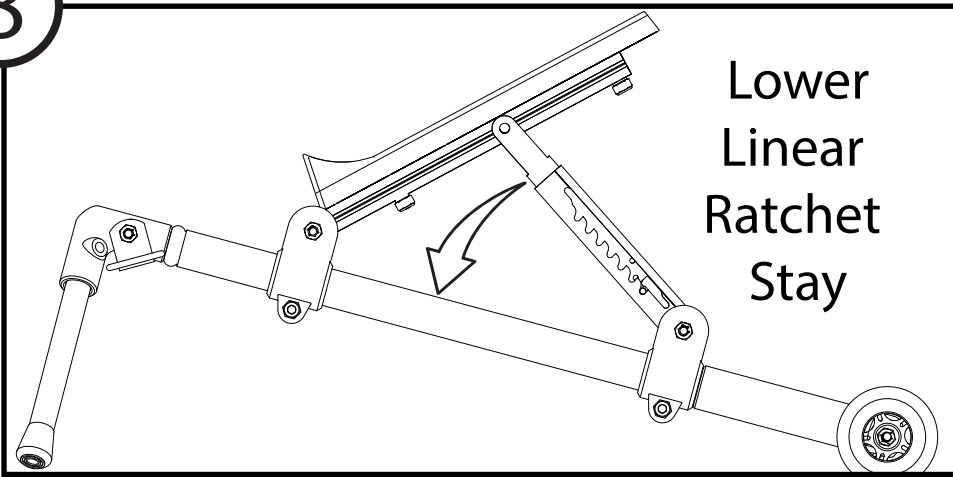
Unlock Buckle and Release Webbing From Chair Leg

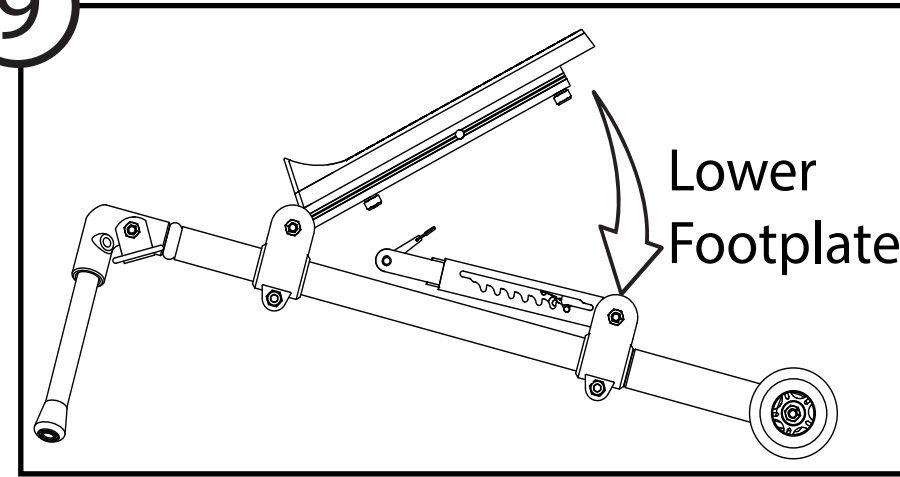
Release Buckle
- 

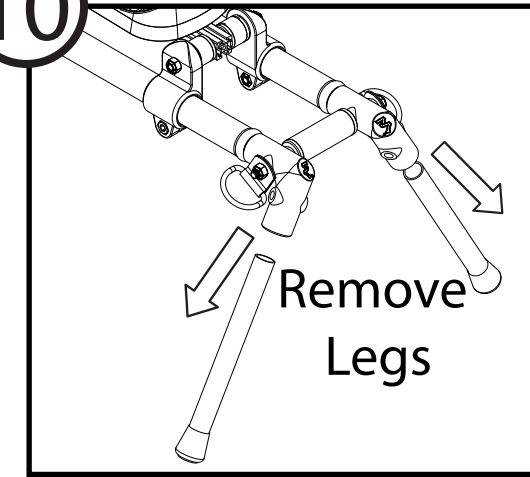
Press to Release

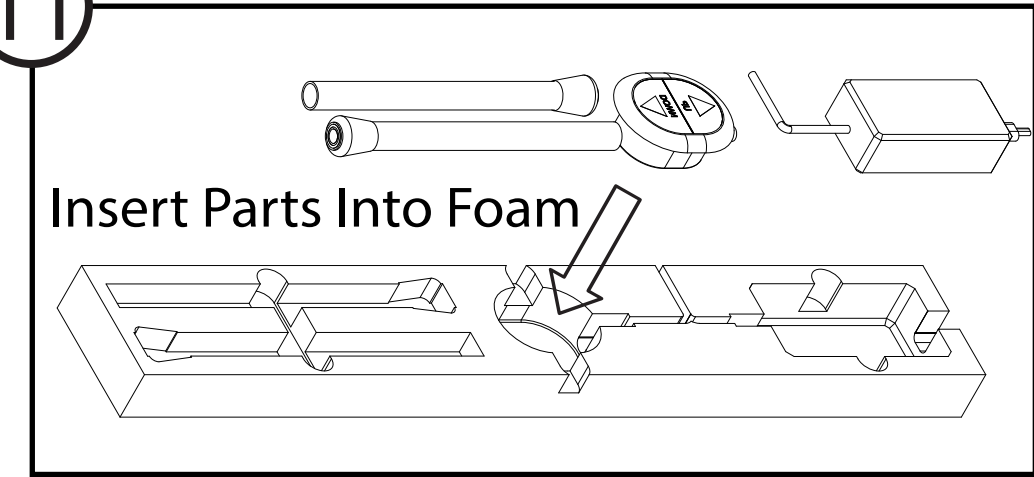
Press to Release
- 

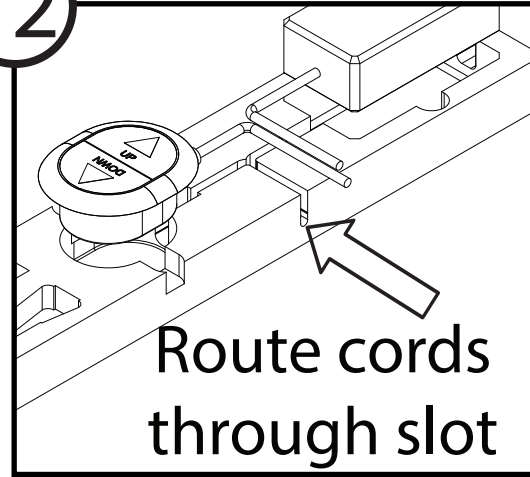
Open
- 

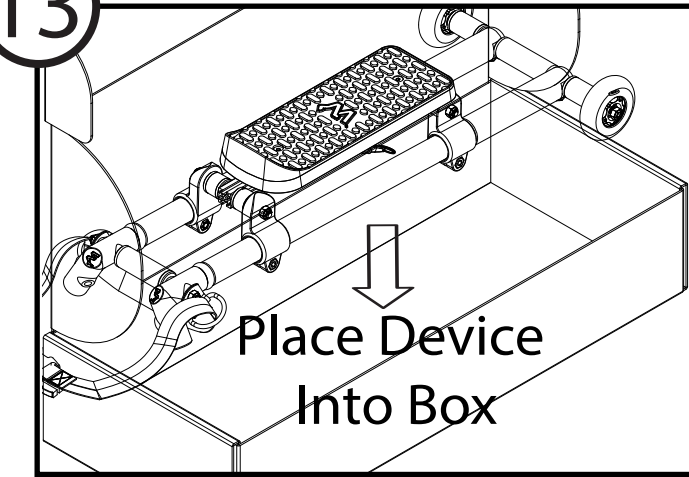
Remove Pin From Footplate
- 

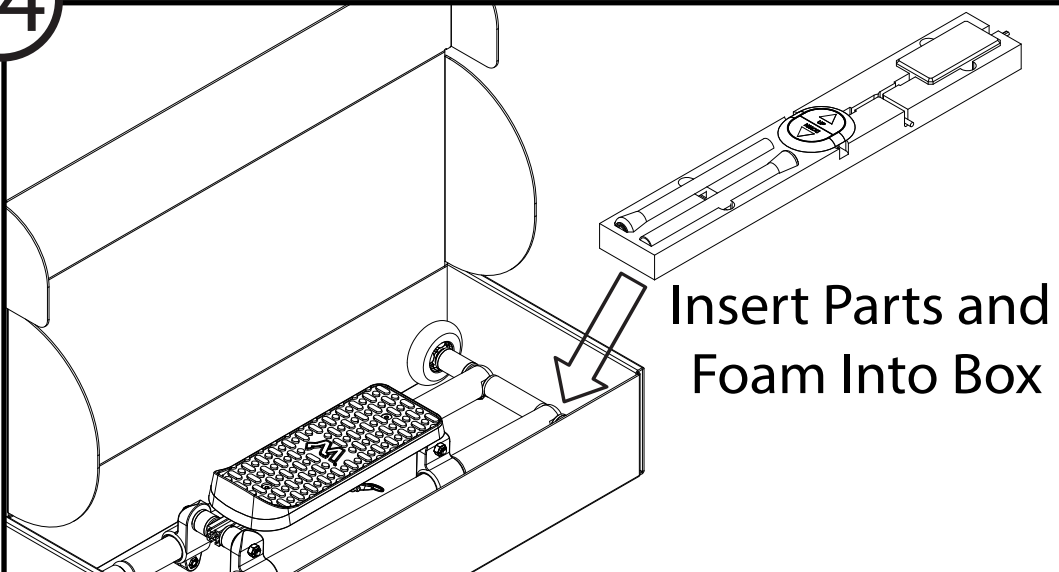
Lower Linear Ratchet Stay
- 

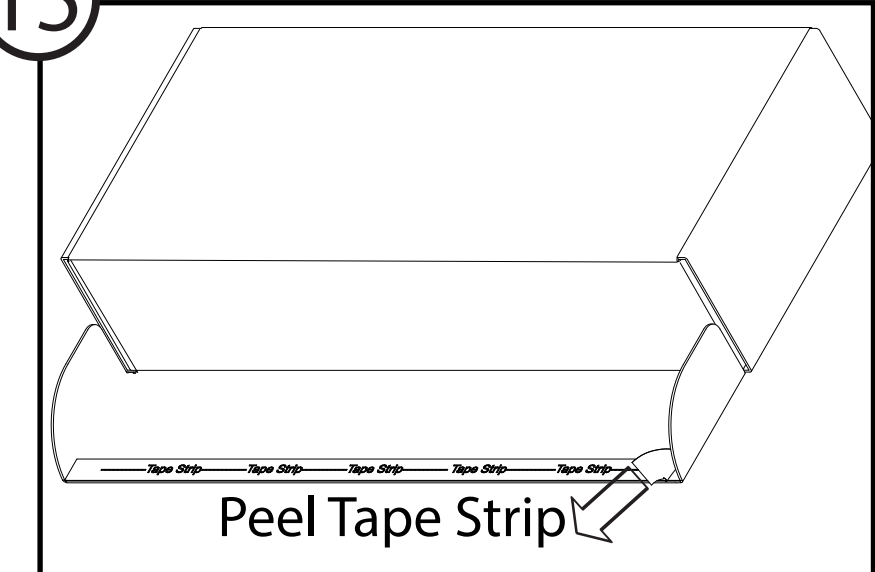
Lower Footplate
- 

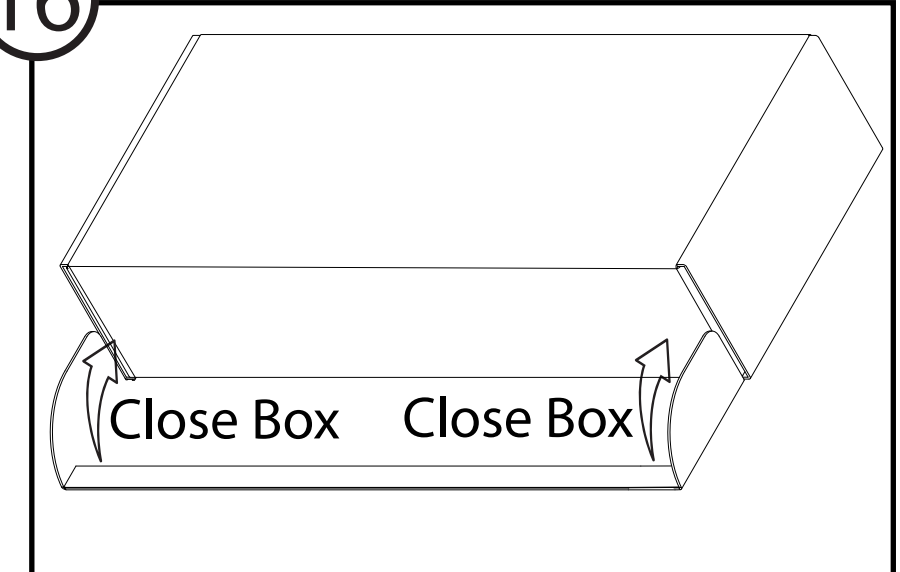
Remove Legs
- 


Insert Parts Into Foam
- 


Route cords through slot
- 


Place Device Into Box
- 

Insert Parts and Foam Into Box
- 

Peel Tape Strip
- 

Close Box Close Box
- 

Peel Off Top Layer of Return Label to Reveal Return Shipping Label
- 

**The UPS Store™** 

Drop off at the nearest UPS Store.